

ESA710, ESA712, ESA715 Software Update 3.7 Instructions

Please follow the instructions for updating your ESA700 series software outlined below.

1. Visit “Software Downloads” page and filter to “Electrical Safety Analyzers”

<https://www.flukebiomedical.com/support/software-downloads>

2. Click “Download” next to ESA710 Software update 3.7.

The screenshot shows the 'Software Downloads' page on the Fluke Biomedical website. The page is filtered to show 'Electrical Safety Analyzers'. Under the 'ESA710' section, there is a card for 'ESA710/ESA712/ESA715 Software Update 3.7'. A red arrow points to the 'Download' button on the right side of this card. The card also includes a description of the update and a note about the file size (886MB).


3. Prepare the file:

- a. Copy the file named **“.biox”** to a USB stick with minimum of 1 GB storage.
- b. Do not rename or modify the file in any way.
- c. The .biox file must be the only file on the USB stick in order to execute the update.

4. Plug the USB stick into one of the two USB-A ports located on the side of the ESA710.



5. On the ESA710 home screen, navigate to:
 - a. Tap the menu icon (☰) located in the top left corner of the screen.

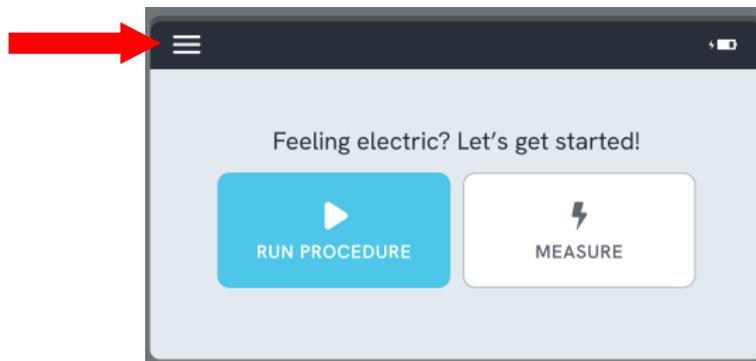

- b. Tap "Settings" -> "About" -> "SYSTEM UPDATE"
 - c. Follow the on-screen instructions to begin the update.

