

# Manual Supplement

Manual Title:	Ansur Test Executive Users	Supplement Issue:	1
Part Number:	FBC-0001	Issue Date:	7/19
Print Date:	April 2008	Page Count:	1
Revision/Date:	6, 3/17		

---

---

This supplement contains information necessary to ensure the accuracy of the above manual.

## Change #1, 737

On page 2-6, add note and change step 6 and replace steps 6-8 with steps 6-9:

### *Note*

*The FTDI Windows driver package is required for communication between Ansur and Fluke Biomedical products.*

6. When the FTDI CDM Drivers window appears, click **Extract**, accept the license agreement and click **Next**.

The package will install automatically.

7. Select **Complete** and click **Next**. Alternatively, select **Custom** to change the destination folder or to remove samples from the installation.

Fluke Biomedical recommends the **Complete** option.

8. Click **Install** to begin the installation. A progress bar indicates the installation process.
9. Click **Finish** to complete the installation process.