IDA-6

Infusion Device Analyzer

Quick Guide

Getting started

There are two ways of working with the IDA-6:

- **Run a procedure.** A procedure is a set of measurements and tasks that are defined in advance, using OneQA on a PC. Visit [OneQA.com](https://www.oneqa.com) to learn more.
- **Perform a measurement session.** Select between three measurement types: Flow & volume, Occlusion, or Bolus & PCA. Add set values and limits to get a pass or fail result.

Measurement setup

With IDA-6 you can test up to four infusion devices at once.

1. Connect the included power adapter and turn on the product.
2. Connect a drain tube to the outlet port of a channel. Lead the other end of the drain tube to a sink or leave it in a container on the table to collect the fluid.
3. Connect an infusion device to the inlet port on the front side of the product. Use distilled or deionized water.
4. Let the infusion device fill the administration set and the channel with fluid. The internal volume of the channel is about 1 mL.

Want to learn more?

You can read the user manual on the IDA-6 screen. From the menu, select **Help** to open the user manual.

Visit [flukebiomedical.com](https://www.flukebiomedical.com) for more information, such as user manuals, specifications and instruction videos. On the website you will also find tips on which accessories you can use with IDA-6.

Safety information

IDA-6 is intended for measurements and tests on medical infusion devices, carried out by trained staff.

Please read the following safety information carefully before using the IDA-6, the product.

Warning

- Use the product only within its specifications.
- Do not use the product on any device that is attached to a patient.
- Do not use administration sets, test tubing or syringes with the product, if they have been or will be in contact with patients.
- Use this product indoors only.
- Only qualified service personnel should service the product.

Caution

- Only use distilled or deionized water with the product.
- Remove internal water from the product before shipping or storing. Do not use compressed air to clean out the product.

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