

# Manual Supplement

Manual Title: ProSim 2/3 Users  
Print Date: January 2013  
Revision/Date: 1

Supplement Issue: **5**  
Date: 5/19  
Page Count: 3

---

---

This supplement contains information necessary to ensure the accuracy of the above manual. This manual is distributed as an electronic manual on the following CD-ROM:

CD Title:	ProSim 2/3
CD Rev. & Date:	1, 5/13
CD PN:	4253822

## Change #1, 67225

On page 45 under, **ECG-Performance-Test** replace the **Amplitude Accuracy** with:

**Amplitude Accuracy** .....±(2 % of setting + 0.05 mV), Lead II (Exception: ±5 % for R waves ≤20 ms)

## Change #2, 59

On page 20, under *Cardiac-Output Test Set-up*, add the following note under the first paragraph.

*Note*

*When the cardiac output simulation is performed, make sure the invasive blood pressure (IBP) cable is not connected to ProSim 2/3.*

## Change #3, 208

On page 43, under **General Specification** replace the **Power** with:

**Power**..... Two 9-V alkaline batteries (IEC 6LR61, NEDA 1604A). Optional battery eliminator:  
15 Vdc, 2.0 mA

## Change #4, 340

On page 33, Table 13, change:

From:

Action	Command
Ventricular fibrillation , coarse	VFB1
Ventricular fribrillation, fine	VFB2

To:

Action	Command
Ventricular fibrillation 1, coarse	VFIB1C
Ventricular fibrillation 1, fine	VFIB1F
Ventricular fibrillation 2, coarse	VFIB2C
Ventricular fibrillation 2, fine	VFIB2F

**Change #5, 700, 708**

On page 15, replace the *Note* with:

*Note*

*See the Swan-Ganz Procedure section to learn more on how the do this serial simulation.*

On page 16, in the table, right column with header "BP4" replace:

From: Swanz-Ganz

To: Swan-Ganz

On page 32, Table 13, change:

From:

<b>Supraventricular</b>	
Atrial fibrillation, coarse	AF1
Atrial fibrillation, fine	AF2

To:

<b>Supraventricular</b>	
Atrial fibrillation 1, coarse	AFC1
Atrial fibrillation 1, fine	AFF1
Atrial fibrillation 2, corse	AFC2
Atrial fibrillation 2, fine	AFF2