

Manual Supplement

Manual Title: SPOT Light Users
Part Number: 4151274
Print Date: February 2012
Revision/Date: 1

Supplement Issue: **6**
Issue Date: 9/18
Page Count: 2

This supplement contains information necessary to ensure the accuracy of the above manual.


Change #1, 63286, 218


On pages 14, under **Specifications** remove **Battery Charger, Safety Standards, Certifications** and **Electromagnetic Compatability** and replace with:

- Safety**..... IEC 61010-1: 300 V CAT II, Pollution Degree 2
- IP Rating**..... IEC 60529: IP20
- Electromagnetic Environment** IEC 61326-1: Basic
- Battery Charger** 100 V to 240 V, 50/60 Hz input, 6V/3 A output. For best performance, the battery charger must be connected to a properly grounded ac receptacle.

Change #2, 456, 458, 614

On page 9, add the following to the end of the page:

By default, the unit automatically turns off if inactive for 5 minutes. To change this setting push and hold  for 3 seconds when the 'Test' field is highlighted. This shows an auto-off option. Select Y for Yes or N for No.

To restore the display to English, turn off SPOTLight, then push and hold  and turn on SPOTLight.

Change #3, 475

On page 5, replace the AC/DC Power Supply with:

AC/DC Power Supply	4766589
--------------------	---------

Change #4, 225, 614

On page 15, replace the **Heart Rate** with:

Heart Rate

Rates30, 40, 60, 65, 80, 100, 120, 140, 150,180, and 240 BPM